Curved Applicator Tip

Curved Applicator Tip

ENG
Re-order No. 934210 (10 cm x 10 G) Re-order No. 934208 (8 cm x 10 G)

ENG
INSTRUCTIONS FOR USE
This Applicator Tip is Supplied Non-Sterile. Do Not Use The Tip If Damaged.
DESCRIPTION: Reusable Curved Applicator Tip, Stainless Steel, 10G.
INDICATIONS: This Applicator Tip is intended for use for delivering hemostatic agents that can be mechanically mixed and delivered to bleeding sites.
DIRECTIONS FOR USE:

- The Applicator Tip must be steam sterilized prior touse (see Cleaning/Decontamination and Sterilization).
- Carefully attach the Applicator Tip to a luer-lock syringe containing the premixed hemostatic agent. Make sure that the luer connector is locked.

DESCRIPTION: Reusable Curved Applicator Tip, Stainless Steel, 10G.

INDICATIONS: This Applicator Tip is intended for use for delivering hemostatic agents that can be mechanically mixed and delivered to bleeding sites.

- DIRECTIONS FOR USE:**
- The Applicator Tip must be steam sterilized prior touse (see Cleaning/Decontamination and Sterilization).
 - Carefully attach the Applicator Tip to a luer-lock syringe containing the premixed hemostatic agent. Make sure that the luer connector is locked.

- CLEANING/DECONTAMINATION:**
- Disconnect the Applicator Tip from the syringe.
 - Rinse blood and other debris from the Tip under cold tap water.
 - Using a luer-lock syringe, flush the lumen of the tip with at least 100 mL of warm water.
 - Clean the lumen of the Tip using a lumen brush.
 - Soak Tip in an enzymatic cleaning solution such as Enzol[®] as per manufacturer's instructions.
 - Be sure all tissue and debris is thoroughly removed from the Tip.
 - Flush the lumen of the Tip with at least 100 mL of water.
 - Rinse Tip thoroughly under running water.
 - Dry Tip thoroughly. Lumen can be dried with compressed air.
 - Pass to processing area for sterilization.

STERILIZATION: The Applicator Tip may be autoclaved up to 20 times using steam sterilization. Using the regular cycle, steam autoclave at 270 °F (132 °C). For flash sterilization, place the unwrapped Tip on a towel in the autoclavable pan/tray, keeping the Tip from touching other instruments. Sterilize at 270 °F (132 °C) for 10 minutes. Flash autoclaving temperature should not exceed 275 °F (135 °C).

Ethylene Oxide sterilization is not recommended.

STERILIZATION: The Applicator Tip may be autoclaved up to 20 times using steam sterilization. Using the regular cycle, steam autoclave at 270 °F (132 °C). For flash sterilization, place the unwrapped Tip on a towel in the autoclavable pan/tray, keeping the Tip from touching other instruments. Sterilize at 270 °F (132 °C) for 10 minutes. Flash autoclaving temperature should not exceed 275 °F (135 °C).

Ethylene Oxide sterilization is not recommended.

STERILIZATION: The Applicator Tip may be autoclaved up to 20 times using steam sterilization. Using the regular cycle, steam autoclave at 270 °F (132 °C). For flash sterilization, place the unwrapped Tip on a towel in the autoclavable pan/tray, keeping the Tip from touching other instruments. Sterilize at 270 °F (132 °C) for 10 minutes. Flash autoclaving temperature should not exceed 275 °F (135 °C).

Ethylene Oxide sterilization is not recommended.

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