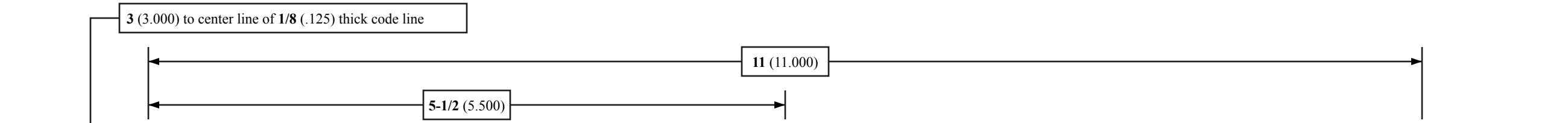
Index Description
Endoscopic Applicator

Endoscopic Applicator

EN INSTRUCTIONS FOR USE
DESCRIPTION: The Endoscopic Applicator is a sterile, single-use applicator device. The Endoscopic Applicator consists of two components: (1) a non-reflective composite cannula, and (2) a composite stylet.
INDICATIONS: The Endoscopic Applicator is intended for use in delivering hemostatic agent to bleeding surgical sites through a 5 mm or larger trocar.

- WARNINGS/PRECAUTIONS:
- Read the hemostatic agent instructions for use prior to using this device.
- Do not use if damaged.
- Do not use air as a propellant to extrude hemostatic agents instead of the stylet.

- DIRECTIONS FOR USE:
1. Single use only - see warnings/precautions.
2. Read the hemostatic agent instructions for use prior to using this device.
3. Transfer both cannula and stylet (obturator) to the sterile field.
4. Attach the syringe with the hemostatic agent to the cannula, ensuring the luer connector is locked.
5. Load the Endoscopic Applicator with the desired amount of hemostatic agent by depressing the plunger on the hemostatic agent syringe.

Single-use, one patient device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of patient.
USA: Rx only
Baxter is a trademark of Baxter International Inc.

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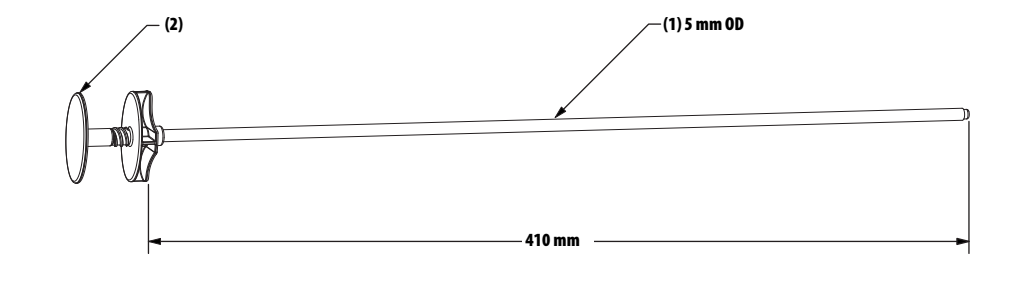
EU Representative: MDSS GmbH, Schifflgraben 41, D-30175 Hannover, Germany

DIST
Distributed by: Baxter Healthcare Corporation, Westlake Village, CA 91362 USA

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
GENERAL NOTES:
1. Adobe Illustrator CS4 artwork file format.
2. Finished part 11" x 17" sheet of paper printed front and back, "M" folded, then folded in half to 5-1/2" x 4-1/4" for a total of 16 panels (8 front, 8 back) with English panel cover.

ARTWORK RELEASE BLOCK
Language: Multilingual—13 languages
Electronic Proofreading Approvals
Manual Proofreading Approvals
1st Proofread
2nd Proofread

Language Legend
EN English
FR French
DE German
ES Spanish
IT Italian
NL Dutch
SV Swedish
NO Norwegian
DA Danish
FI Finnish
PT Portuguese
EL Greek
CS Czech