

Malleable Tip Endoscopic Applicator Indication

The Malleable Tip Endoscopic Applicator is intended for use in delivering a hemostatic agent to bleeding sites through a 5 mm or larger diameter trocar.

Malleable Tip Endoscopic Applicator Important Risk Information

Read the hemostatic agent instructions for use prior to using this device.

Do not use if damaged.

Do not use air as a propellant to extrude hemostatic agents instead of the stylet.

Do not use Malleable Tip Endoscopic Applicator to manipulate or retract organs or tissue.

Do not bend the malleable tip against organ or tissue.

The Malleable Tip Endoscopic Applicator should only be used by persons adequately trained in Minimally Invasive surgical procedures.

Do not reuse or resterilize. Reuse of single- use devices creates a potential risk of patient or user infections. Contamination of the device may lead to injury, illness or death of the patient.

Rx Only: For safe and proper use of this device, please refer to full device Instructions for Use.